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EN Cohesive bandage to prevent injury to extremities (especially hands and fingers) and to prevent the ingress of dirt and water.

Instructions for use:

Rip the bandage off and rotate around the area to be protected with slight tension. For fingertip applications, wrap the product once around the finger with approx. 1.5 cm overstand. Fold the excess product over and then wrap the rest (at least 2 layers) of the bandage around the finger.

Storage: Store unopened in original packaging in a cool and dry place. Recommended storage temperature 10°C to 25°C. Shelf life: 5 years in the original container.

Protective measures: In order to avoid congestive effects, the bandage should be wrapped around the area using only slight tension, and it must not come into direct contact with a wound. Do not use the product in the vicinity of rotating parts and do not wear at night. Do not swallow. Keep out of reach of children. Remove bandage if uncomfortable.

Residual risks: Danger of blood congestion due to too tight wrapping. Risk of feeding or capturing when used close to moving or rotating parts. Danger of suffocation if the bandage or packaging is swallowed. Risk of discomfort due to long exposure possible. If pressure is uncomfortable, the bandage must be removed.

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Nohtec GmbH